



Andro-Penis® Product Brochure

(UK) www.andro-penis.co.uk

(Europe) www.andropenis.net

(Rest of The World) www.andro-penis.net

(Other Products) www.virilityhealth.com

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com |



Andro-Penis®	3
Effectiveness and Guarantee.....	3
Clinical Proof.....	10
Results.....	13
Frequently Asked Questions.....	14
Certifications.....	19
World Penis Sizes.....	21
Price.....	22
Ordering Information.....	22
Order Form.....	24
Other Virility Health Websites.....	25

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |



Andro-Penis®

You can have a permanently larger penis!

Andro-Penis® - Lowest Prices on the Net!

This is a Medical grade penis enlarger used by doctors, urologists and clinics around the world.

- Permanent results.
- No surgery or medication required.
- Patented internationally.
- Andro-Penis is manufactured using high quality medical materials.
- Andro-Penis can be worn while walking, sitting or standing.
- Also used for treatment of penis curvatures and Peyronie's disease.

Effectiveness and Guarantee

The Andro-Penis offers three Guarantees



- A medical guarantee, since it is approved by the Health Authorities of the European Union, and is registered at the Spanish Ministry of Health. It holds the CE Certification that classifies it as a Type 1 medical device, which excludes any side effect, both on the short and on the long term.
- A guarantee based on results. If in two months you cannot get used to the device or the method, you do not like it or for whatever other reason you decide to send it back to Virility Health Ltd, we will refund you 50% of the total amount you paid for the device (we withhold 50% of the amount to cover shipping and sterilisation costs). All you have to do is send the device in its original box and with all its metal parts to Virility Health Ltd. Refunds are transferred to your bank account at the end of the following month after Virility Health Ltd receives the device.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com

Effectiveness 97%

- A Scientific Guarantee, since very prestigious urologists at international level have carried out scientific studies that prove an effectiveness of 97.5%

Please take the time to read the information below and via the links so you can truly understand why and how this product works. You will not see any wildly exaggerated claims, just the facts presented cleanly and simply.

Andro-Penis® - Medical Grade Penis Enlarger

Medical Warranty This is a medical device devised by Dr Gomez de Diego MD, It is approved by the Health Department of the European Nations, and the German Ministry of Health. It holds the CE Certification that classifies it as a medical device Type 1 which assures the absence of short or long term secondary side effects. It fully displays the the European certification CE, awarded by the European ministry of Health. It's Medical Health permit number is 5144-PS.

Scientific Warranty

Since very prestigious Urologists from around the world have performed scientific studies that corroborate the Andro-Penis's effectiveness of 97.5%. Andro-Medical's own tests claim 95% effectiveness. This is an incredibly effective device.

An incredible medical breakthrough The Andro-Penis® penile enlarger makes it truly possible to have a longer and thicker penis Permanently without surgery. This is a registered medical device used by urologists, doctors and clinics worldwide and Harley Street.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

You can have a permanent increase in length of up to 5 centimetres and an increase in width diameter of 1 centimetre. The Andro-Penis® works by the principle of traction as applied in modern medicine, for the generation of new tissue to cover burn wounds or areas of hair loss or for the lengthening of bones. The same principle is used in other cultures to lengthen different parts of the body such as the Giraffe women of the Paduang tribe in Birmania.



Based on this principle the Andro-Penis® was designed. It is able to exert a gradual traction force of 600-1500 gms.

Using the Andro-Penis® for 4-10 hours a day for 3-6 months you will achieve:

- An increase in the length of the penis in erect and flaccid state
- An increase in girth in erect or flaccid state



(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com

Andro-Penis® - Medical Pedigree

The only penis enlarger endorsed by the medical community

Thousands of physicians worldwide have validated the efficiency of the Andro-Penis® in their clinics and hospitals. The highest quality medical grade materials are used in its manufacture entirely suitable for continuous contact with the skin and mucous areas.

Andro-Penis® - How does it work?

Traction

The Andro-Penis® applies a traction force of 600 to 1500 grams of weight upon the penis during continuous periods of time. Its vector is located on the shaft of the penis.

This traction force provokes a reaction of adaptation at a structural level on the tissular components of the penis. The result is an increase in cellular multiplication that translates into the following:

At a microscopic level, An increase in the total number of cells.

At a macroscopic level, A permanent increase in the total length of the penis in its erect and flaccid state. A permanent increase in the perimeter of the base of the penis in its erect and flaccid state. The abilities of erection and reaching an orgasm are in no way affected by the treatment..

The Andro-Penis® should be used for 10 hours a day during a period of 4-6 months. The device is fairly small and inconspicuous and should not interfere with your daily routine.

Andro-Penis® - How do you use it?

Using the Andro-Penis®

This device is easy to place and remove at any time. Since the product is discreet and comfortable, it can be worn while walking, sitting or standing.

The device is placed while the penis is in the flaccid state and should be worn between 4 to 9 hours a day with short breaks. You should notice an increase of 1/2cm per month.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

The product should be worn for a period of 4 to 6 months according to the size the patient wishes to attain. After treatment the results will be permanent.



Andro-Penis® - What do I get for my money?

The product®

The Andro-Penis® comes in a quality cherry varnish wooden box which looks simply stunning.



Inside you will find the Andro-Penis®, a CD Rom, an instruction booklet in English, French, German, Spanish, Dutch, Italian, Portuguese, Greek and Russian.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com



HelpLine

Every Customer gains access to a free medical helpline, over the phone or via e-mail

Andro-Penis® - Product Guarantees

Health Certification

The Andro-Penis® complies with all the quality requirements imposed by the competent authorities in health matters. It holds the CE health marking as type 1 medical device in Europe. The FDA certificate has been applied for in The US and Japan.

Safety

The Andro-Penis® excludes any risk to one's health, has no side effects in the short or long term.

Medical Qualities

The highest quality medical materials are used for the production of the Andro-Penis®, entirely suitable for continuous contact with the skin and mucous areas.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com



Quality System

Andromedical® has implemented a Quality management system in accordance with the UNE-EN-ISO 9001:2000 standard for production and commercialisation of its products, certified by the British Standards Institute (BSI)

Click here to see the certificate in a new window

Patent

The Andro-Penis® is patented internationally. It's industrial property rights are duly protected.

Andro-Penis® - About Andromedical®

Company Philosophy

Andromedical® is a European Medical Company and is established internationally.

In 1997 Doctor Eduardo Gomez de Diego set up the Biomedical Andrology and Aesthetics clinic. The high volume of requests from men desiring a larger penis lead to the creation of Andromedical in the year 2000 and ultimately this amazing medical breakthrough.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

Clinical Proof

Clinical Trial.

Number of patients: 37 patients aged between 22 and 60 years of age. These men taken from different cities in Spain.

Selection of patients: Patients included were healthy men with normal erection capabilities and without penile surgeries. Patients with penile or other diseases were excluded from the study.

Device used: The Andro-Penis® A German device to exacting German engineering standards and certified medical grade.

Traction force: 600grams during 1st month 900grams during 2nd month, 1100grams during 3rd and 4th month and 1200grams during 5th and 6th months.

Usage period: 10 hours a day every day for 3-6 months.



Results.

The increase in the length of the penis in erection is relative to the time in which the device is worn. Such growth is lineal as observed in the chart. This translates to the longer you wear the device the more length is obtained. The lineal correlation coefficient between time of use and increase in length of erection is of 0.760.

The average increase in the length of the penis in erection by month is 0.4726cm. Nearly half a centimetre a month
The confidential interval of 95% is of [0.4283;0.5169] which indicates a minimal gain in the population of 0.4283cm/month.

Age makes no difference

A very interesting result was that the increment in the length does not depend on the age of the patient, since the coefficient of the linear correlation is not significant. So you can benefit from the Andro-Penis® no matter what age you are.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com



3 Months usage

The median average increase for men who used the device for 3 months was an increase in erect state of 1.4118 cm A median increase of 10.5580% of the original size. Imagine having a penis 10% longer in just 3 months!!

6 Months usage

The median average increase for men who used the device for 6 months was an increase in erect state of 3.3333 cm A median increase of 27.5% of the original size. Imagine having a penis over a quarter as long again in just 6 months!!

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

The complete trial chart

Name	Age	Start	L-Erec1	L-Flac1	P-Erec1	P-Flac1	Month	L-Erec2	L-Flac2	P-Erec2	P-Flac2
LAA	60	15.1.98	12.0	8.0	12.0	10.0	3.0	13.5	9.0	13.5	10.5
EAG	37	30.1.98	14.5	8.0	12.0	8.5	4.0	16.5	9.5	13.0	10.5
VA	27	26.11.97	16.3	10.4	13.9	10.1	3.0	17.8	12.5	14.5	11.5
EAA	46	15.3.98	14.5	10.5	12.0	10.0	6.0	17.6	15.7	13.2	11.2
JBV	25	27.3.98	15.0	8.0	13.0	9.0	3.0	16.6	9.5	13.3	10.5
ABB	39	29.6.98	14.0	11.0	13.0	11.0	3.0	15.5	12.5	14.0	12.0
CBG	37	19.1.98	12.5	6.0	12.0	9.0	4.0	14.0	8.8	12.5	9.4
JBL	25	7.5.98	13.7	9.0	11.5	10.0	4.0	16.3	11.5	13.5	12.0
JCB	27	19.11.97	13.0	8.0	14.5	12.0	6.0	16.4	11.1	14.2	12.2
JICA	33	1.6.98	10.5	4.9	11.0	9.5	3.0	12.0	5.5	11.5	9.5
JCA	32	4.2.98	14.0	10.0	10.0	9.0	4.0	15.9	12.0	12.2	10.5
ODV	25	4.3.98	16.5	9.5	13.0	9.7	3.0	18.0	11.0	13.3	10.0
PDS	22	10.6.98	14.4	8.3	11.0	7.8	3.0	15.8	9.0	11.6	8.3
AGM	41	26.11.98	13.0	9.0	11.0	10.0	4.0	15.0	9.5	11.0	10.0
MGF	32	29.9.97	12.5	5.5	12.5	10.0	4.0	14.5	8.0	13.0	10.0
AHM	44	5.3.98	11.5	8.0	13.0	12.0	3.0	13.0	9.5	14.0	12.0
AAMP	37	12.3.98	12.7	7.0	10.5	7.5	3.0	14.0	9.0	11.0	9.0
JLMO	34	30.1.98	14.8	11.0	11.0	9.3	4.0	15.8	11.0	12.7	9.3
JAMV	41	28.7.97	17.0	10.0	14.0	12.5	5.0	19.6	12.3	15.0	13.5
FOR	30	9.11.97	12.5	7.0	10.0	9.0	5.0	14.5	11.0	11.0	10.5
ROM	28	12.11.97	16.0	8.5	13.0	9.0	5.0	18.0	12.5	13.5	10.5
JPC	33	16.1.98	13.7	7.2	12.3	10.1	3.0	16.7	8.7	13.1	10.9
JAPG	29	4.11.97	10.0	8.0	12.0	10.0	6.0	13.5	11.5	13.0	11.0
FPR	30	20.3.98	10.5	7.0	12.0	10.0	3.0	11.3	7.3	12.7	10.0
JPF	42	23.3.98	13.0	7.0	13.0	10.0	3.0	14.5	8.5	13.0	10.5
AJRF	26	28.11.97	14.0	9.0	13.0	9.0	4.0	16.0	10.7	13.0	10.0
ARR	58	7.10.97	11.0	7.0	11.0	9.0	3.0	12.0	8.0	12.0	10.0
RRG	25	25.11.97	14.5	11.0	11.0	10.0	3.0	15.0	12.4	11.7	10.8
CSM	35	24.2.98	15.0	9.0	11.0	9.0	3.0	16.5	11.0	12.0	10.0
ASE	35	20.3.98	12.5	7.5	11.5	9.0	4.0	14.5	9.5	12.5	10.5
ASPA	42	7.11.97	14.0	7.5	12.0	9.0	4.0	16.0	9.5	13.0	10.5
SSF	27	15.8.97	14.5	7.0	14.5	8.0	3.0	15.9	8.0	15.0	8.7
ISB	22	23.9.97	11.5	7.5	11.0	9.5	4.0	13.5	10.2	11.3	10.3
FT	53	19.11.97	14.5	10.0	13.0	10.0	3.0	16.0	11.5	13.5	10.5
EVC	24	29.12.97	11.0	7.0	11.5	9.0	5.0	13.5	9.0	12.5	10.5
PV	33	8.10.97	12.0	8.5	15.5	13.0	4.0	13.0	9.5	16.0	13.0
JSVS	32	24.1.98	12.0	6.0	10.5	9.0	4.0	14.0	8.5	12.5	9.5

L-Erec1 = Length in Erection initial
P-Erec1 = Perimeter in Erection initial
L-Erec2 = Length in Erection final
P-Erec2 = Perimeter in Erection final

L-Flac1 = Length in Flaccidity initial
P-Flac1 = Perimeter in Flaccidity initial
L-Flac2 = Length in Flaccidity final
P-Flac2 = Perimeter in Flaccidity final

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

Conclusions

The use of the Andro-Penis® will increase the length of the penis in both erect and flaccid states

- The increase in size is proportional to the time used.
- The increase in length is not dependent upon the original size of the penis
- For 95% of patients the average growth in length erect was 0.4283 and 0.5163 cm per month used!!
- Age is not important

Results

The purpose of this treatment is to stimulate the growth of the tissues of the penile shaft. This translates to:


- Permanent lengthening of the penis of **3 to 5 cm** in erection. One inch equals **2.5 cm**
- Also **3 to 5 cm** in flaccidity
- And a thickening of **1 cm** in girth.

The device is worn in the flaccid state for a period of either **4 or 10 hours** a day obtaining an accumulative growth of **0.5cm per month**. It should be worn for a period of **4-6 months** depending on the growth desired. The centimetres gained will last a lifetime

There is a **correction of 70%** in penile deviations or curvatures. The treatment will last 6-8 months.

Before and After Photos

The two series of photos taken before and after 6 months of treatment, show the efficacy of this method.

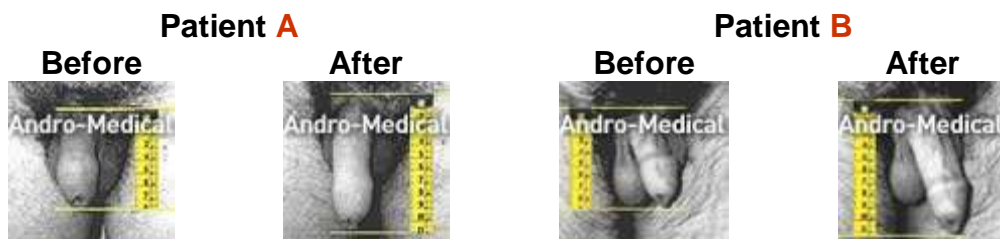

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
 | Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
 | www.virilityhealth.com |

Frequently Asked Questions

1) What is the foundation behind this principle?

It is based in the medical principle of traction, that when executed in a controlled, gradual and progressive Manner on the penis will stimulate the growth of the suspensory ligament, corpus cavernosum, and the rest of the penile tissues.

2) Has the principle of traction been used in medicine before?

Yes. In modern medicine this principle of traction is used to create new tissue for skin grafts, for the elongation of bones and to create new skin with hair to cover bald areas. Also this principle has been used in ancient cultures to create long necks like in the Giraffe women of Birmania and to create longer lips in certain African tribes.

3) But, can the penile tissues grow? Is the penis made of bone or muscles?

The penis is a conglomerate of elastic tissues and is devoid of muscles and bones making it easier for this tissue to respond and grow when placed in traction.

4) What is the maximum growth in centimetres that I can obtain?

It all depends on the individual, approximately 4 centimetres. One inch equals 2.5cm

5) Which is the minimum growth that can be obtained?

The minimum growth is of 1.5 cm. The average growth is 3 cm, based on 95% of the patients.

6) Is there growth both on erection and flaccidity?

Yes, the growth is proportional and it will be evident in erection as well as in flaccidity.

7) Will it help to correct penile curvatures?

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |



Yes. In a period of 6 to 7 months of treatment we are able to correct 70% of the curvature. There will be a marked improvement.

8) Are the results permanent?

Yes, because there has been growth of the penile tissues.

9) Will the glans also grow?

No. Only the penile shaft. The sensitivity of the glans will not be affected.

10) Will the lengthening make the penis thinner?

No. The growth is proportional throughout the whole penis. There will be an increase in length as well as in girth since the new tissue formed creates more volume and occupies space.

11) Any side effects?

No. This method has no collateral or side effects as long as you follow the instructions.

12) Can it affect the erection process or cause impotence?

No, the erections will not be affected and the sexual drive is improved since the patient's self esteem is increased due to a larger penis.

13) Can I do the treatment on my own?

Yes, as long as you follow the instructions provided.

14) Is this treatment only for small penises or for any other size?

This treatment can be used to enlarge small penises as well as to increase the dimensions of medium and large ones.

15) Is there a minimum penile length required to be able to proceed with the treatment?

Yes, the minimum length in erection must be that of 8cm. For penises less than 10cm in length the Mini extensor will be used.

16) How long is delivery?

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

The device is shipped from stock on receipt of cleared funds within 24 hours Monday - Friday.

17) Which is the ideal age to begin the treatment?

From 18 years on. The treatment can be done in older men. Our research included men of 60 and 70 years of age and it showed that traction will stimulate the growth of tissues at any age.

18) How is this different from the vacuum pumps?

Vacuum pumps work by suctioning the tissues to obtain an immediate and abrupt vasodilatation. These pumps can help in attaining an erection but are not as effective in generating permanent long term penis enlargement.

19) When I am done with the treatment, can someone else use the device?

It is not advised to share the device with others.

20. Is there any pain?

a. No, this product is a mechanical device and is painless. The user at first may experience the presence of product, but in time this sensation will soon disappear.

21. Should I remove product when urinating or defecating?

a. Yes, when finished replace back onto the penis.

22. Should I remove product before having sexual intercourse?

a. Yes, waiting one hour before replacing it back onto penis.

23. Is blood circulation cut in penis?

a. No, follow instructions carefully in order to avoid discomfort.

24. How can I use product without my family finding out?

a. Use it during working hours or when out of home.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

25. Can it be worn at night?

a. Yes, although many patients wake up with product dislocated owing to the amount of times the user moves at night or nightly erections. In these cases, the product should only be worn 3 hours per night. We recommend that the product shouldn't be worn at night during the first few days of usage or until you feel totally comfortable with handling the product.

26. If I have an erection with the stretcher on, what will happen?

a. Nothing will happen. You'll feel a little pressure, but it should be painless.

27. Can I use the stretcher with a bathing suit?

a. Yes, if you use a loose fitting bathing suit (Trunks for example.) with a net crouch, you can usually wear the stretcher without notice.

28. Can I use the stretcher at the airport?

a. We recommend that the stretcher be removed at the airport.

29. Can I wear jeans or tight-fit clothing while using the stretcher?

a. You may wear jeans that are not too excessively tight.

30. If I am sat for a long length of time, can I use the product?

a. Yes, we will tell you how to modify the stretcher for these types of cases.

31. Will the stretcher rust if exposed to water? Can it be washed and how?

a. This product will not rust. European Union Certification guarantees that product is free of defects and guarantees that product has been inspected up to par. Stretcher can be washed once a week with mild soap and water.

32. Must product be oiled or greased?

a. At first it is not necessary, but we do recommend that you apply small amounts of oil on the bars in order for it to slide up and down with ease.

33. When treatment is finished, must I return product?

a. No, the stretcher now belongs to you.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |



34. What is the product made out of? Does it need batteries? Does it produce electrical shock?

a. This product is totally mechanical. It is used to stretch the penis in its un-erected state. It does not require batteries nor electricity.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

Certifications

Health Ministries and Dept. Certifications

EUROPE: Certification CE

The **Andro-Penis**® has been classified by the European Union Health Authorities as a Medical Device Type 1, and has the CE sign. (Tests of approval were realized as established in the declaration of Helsinki, adopted in 1964 during the 18th World medical Assembly in Helsinki, Finland, and revised in 1989 during the 41st assembly in Hong Kong).

The **Andro-Penis**® is made according to the standards of the European Union, using only medical parts of the highest quality, suitable for being in continuous contact with the human skin tissue.

Andromedical® is registered under the following Health Certification: D.A.P.S. 334/M, handed out in Europe by the Health Authorities of the Autonomous Region of Madrid, Spain.

MEXICO

The **Andro-Penis**®, is at present, the only medical device of its kind to be approved by the Federal Health Authorities of Mexico and by the Ministry of Health of each State of the Republic. In the State of Jalisco the permit number is: S.S.08Q03N. Our product is manufactured to comply with the highest medical standards and highest product quality that will enable our device to be in direct touch with the skin and mucous membranes for a prolonged period of time.

The **Andro-Penis**® is classified as a Medical Device and obtains the Federal Certificate of Import SSA-04-003. ANDROMEDICAL MEXICO S.A. DE C.V. it is registered in the S.H.C.P. (Secretaría de Hacienda y Crédito Público) with the R.F.C.(Registro Federal de Contribuyentes): AME010326 LC3.

MERCOSUR

The **Andro-Penis**® is classified as a Medical Device. Our product is manufactured to comply with the highest medical standards and product quality that will enable our device to be in direct contact with the skin and mucous membranes of humans for prolonged periods of time.

Our product complies with the Health specification of the countries of Mercosur (Argentina, Brazil, Uruguay, Paraguay and Chile) and is in the

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com



process of obtaining the Certification from the Departments of Health of such countries.

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

World Penis Sizes

COUNTRY	AVERAGE ERECTION	AVERAGE FLACIDITY	SOURCE
 USA	12,9 cm	8,8 cm	Wesseells H., Lue T., McAicnich J.
 Germany	14,48 cm	8,8 cm	Dr. Gunther Hagler, Urologist
 Spain	13,58 cm	8,8 cm	Dr. Javier Ruiz Romero Clínic Tres Torres Barcelona, 2001
 France	16 cm	12 cm	Bouchet A., Cuilleret, J.
 Japan	13 cm	8 cm	Japanese Journal of Sexology
 Brazil	12,4 cm		Dr. Paulo Palma, Urólogist Carlos da Ros Claudio Teloken
 Italy	15 cm	10cm	Dr. Jamal Salí del a Sociedad de Andrología de Italia. Dr. Carpenito Ambulatori , Andrologist
 Venezuela	12,7 cm	9,5 cm	"Dimensiones peneanas en la población venezolana" Servicio de Urología, Hospital Domingo Luciani
 México	14,9 cm	No Data	Dr. Francisco Ordoñez Lifestiles Condom Co. In Cancún, Mexico
 Greece	12,18 cm	No Data	Dr. Spyropoulos E. et coll Hospital Naval de Veteranos Atenas 2003
 India	10,2 cm	No Data	"The Jacobus Survey"
 Saudi Arabia	12,4 cm	No Data	Dr. Mohamed Habos. Clínica de Comportamientos Y Enfermedades de los órganos genitales masculinos
 Chile	14 cm	No Data	Dr. Eduardo Pino, Urólogo/Andrólogo Clínica Andromex Santiago de Chile
 Colombia	13,9 cm	8,95 cm	Acuña A., Villalba J. & Juan Carlos Villalba Clínica A. Saio, Bogotá.
 Korea	9,6 cm	6,9 cm	Hwancheol Son, Hanjoon LeeJung-Sik Huh, Soo Kim & Jae-Seung Paick

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
 | Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com



Price

(Please visit website for latest prices)

UK - £236.75 (GBP) including shipping and VAT @17.5%

Europe - €294.99 (Euro) including shipping and VAT @ 17.5%

Rest of the World - \$295.00 (USD) including shipping

Ordering Information

Online

You can place your order online at the following websites

Dedicated Sites

UK - www.andro-penis.co.uk

Europe - www.andropenis.net

Rest of The World - www.andro-penis.net

Virility Health Ltd General Website

UK – www.virilityhealth.co.uk

Europe – www.1stvirilityhealth.net

Rest of The World - www.1stvirilityhealth.com

Instalments

This product can also be paid for in instalments. See any UK website for details (UK customers only)

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com |



Cheque or Postal Order

Please complete our order form below and send it along with your payment to

Orders Department
Virility Health Ltd.
3 Leigh Court Close
Cobham
Surrey
KT11 2HT

Please make your cheque or postal order out to **Virility Health Ltd.**

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
| www.virilityhealth.com |

Order Form

Your Details	
Name	
Address	
Post Code	
Telephone No	
Email Address	

Product Description	Qty	Price Each	Total
Andro-Penis® Medical grade penis enlarger			
Penis Equal or Longer than 10cm when Erect.			<input type="checkbox"/>
Penis Equal or Shorter than 10cm when Erect.			<input type="checkbox"/>
Total			

Payment Method

Cheque / Postal Order Enclosed			<input type="checkbox"/>
Credit Card / Delta / Switch / Solo etc Details			<input type="checkbox"/>
Credit Card No.	Issue No.	Security Code	Expiry Date

Name	Signed	Date



Other Virility Health Websites

To view the full range of Virility Health Ltd's products please visit

UK – www.virilityhealth.co.uk

Europe - www.1stvirilityhealth.net

Rest of The World - www.1stvirilityhealth.com

(Price correct at time of publishing)

Marketing & Sales Dept - Virility Health Ltd. 3 Leigh Court Close Cobham Surrey KT11 2HT
| Customer Service 0800 081 0850 | Fax +44 (0) 870 762 1381 | E: customer.service@virilityhealth.com |
www.virilityhealth.com |